Kent polish

MINT FLAVOUR

fr

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Prophylaxis paste mint flavour

en

Contains only natural flavourings and no fluoride. The blends of abrasives used in the various grades are selected to maximise the removal of plaque and tartar without appreciable abrasion of enamel. The thixotropic properties are carefully controlled to ensure easy transfer to cup or brush without undesirable spattering at the rotation speeds used in prophylaxis.

Indications:

Use for the removal of plaque and tartar from patients teeth.

Directions for use:

The prophypaste is thixotropic, so, if desired fluidity can be increased by gentle spatulation before use. Replace lid after use. For best results, use a prophy type handpiece and rubber cup/bristle brush combination. Fill rubber cup/bristle brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup/bristle brush goes below the free margin of the gingiva into the gingival sulcus. Repeat this procedure for each tooth until all exposed surfaces have been treated. Unwaxed dental floss may be used to remove any excess paste through interproximal areas.

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Advena Ltd., Tower Business Centre,
Advanced Healthcare Ltd