Nanohybrid Composite +

- fr Composite nanohybride photopolymérisable
- en Nano hybrid light-cured composite
- es Composite nano híbrido fotopolimerizable
- it Composito nanoibrido fotopolimerizzabile
- pl Compósito nanoibrido fotopolimerizável
- sk Nanohybridný svetlom vytvrdzovaný kompozit
- مركب نانوي هجين معالج بالتصلب الضوئي
- ru Наногибриден фотополимеризиращ композит
- cz Nano hybridní kompozit vytvrdzovaný světlem
- gr Nano-υβριδική φωτοπολυμεριζόμενη σύνθετη ρητίνη
- ro Compozit nanohibrid fotopolimerizabil

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Nanohybrid Composite +

Light-cured restorative nano-hybrid composite resin

en

General Information

NANOHYBRID COMPOSITE+ is a light-cured restorative nano-hybrid composite resin for posterior and anterior restorations. Nano-hybrid formula delivers superior smoothness for easy polishability and exceptional wear resistance for long-lasting restorations. NANOHYBRID COMPOSITE+ exhibits low polymerization shrinkage, good depth of polymerization and high compressive strength. NANOHYBRID COMPOSITE+ offers various shades, good esthetics, good radiopacity.

Composition

NANOHYBRID COMPOSITE+ contains Bis-GMA and UDMA resins with UDMA consisting over 4wt%. The inorganic filler loading is 70% by volume with a particle size between 16nm and 750nm.

Indication

• Class I-V restorations. • Directive anterior and posterior restorations. • Indirect restorations including Inlays, Onlays and Veneers. • Direct veneers. • Blocking out undercuts. • Repair of composite and ceramic veneers. • Core build-ups. • Splinting mobile teeth

Precautionary Information

1. Check the expiration date and appearance condition of product and package.
2. Follow the application after being well acquainted with the Instruction for use.

Application

Preparation

1. Prophyl: Teeth should be cleaned with pumice and water to remove surface stains.
2. Shade selection: Before isolating the tooth, select the appropriate shade(s) of restorative material.
3. Isolation: Adequate isolation is absolutely necessary for achieving optimum results. The use of an Kent Dental: dental dam is recommended. Place a matrix and an interdental wedge, if necessary.

Restoration

1. Cavity Preparation: Prepare the cavity according to the requirements of the adhesive technique, i.e. protecting the tooth structure. Do not prepare sharp, internal edges and angles or additional undercuts in caries-free areas.
2. The dimensions of the cavity are generally determined by the extent of the caries or the size of the old restoration. Caries-free cervical defects are not prepared, only cleaned with pumice or other suitable cleaning pastes with the help of rubber cups or rotary brushed. Subsequently, remove all residues in the cavity with water spray and dry with water-and oil-free air.
3. Pulp Protection: Apply a Kent Dental calcium hydroxide pulp protector. DO NOT USE EUGENOL-BASED MATERIALS to protect the pulp as these materials will inhibit curing of NANOHYBRID COMPOSITE+.
4. Adhesive System: Follow the manufacturer's instructions, use for example KENTBOND+ Kent Dental and GEL ETCH Kent Dental or Kentbond Universal Kent Dental.
5. Dispensing the composite: Follow the directions corresponding to the dispensing system chosen.
 - a) Syringe: Dispense the necessary amount of restorative material from the syringe onto the mixing pad by turning the handle slowly in a clockwise manner. To prevent oozing of the restorative material when dispensing is completed, turn the handle counterclockwise half a turn to stop paste flow. Immediately replaces syringe cap. If not used immediately, the dispensed material should be protected from light.
 - b) Single-dose capsule: Insert capsule into a universal restorative dispenser. Refer to separate restorative dispenser instruction for full instructions and precautions. Extrude restorative directly into cavity.
6. Placement: For optimum results, apply NANOHYBRID COMPOSITE+, in layers not exceeding 4 mm and adapt with a suitable instrument. Use a special contact point instrument for large cavities where the contacts are difficult to create. Sufficient exposure to the curing light prevents incomplete polymerization. For the recommendations regarding exposure time and light intensity, see following table:

Minium intensity 600 mW/cm ² light		
Shade	Incremental Depth	Cure Time
A1, A2, A3, A3.5, A4, B1, B2, C2, TW	2.0 mm	20 sec.
A02, A03	1.5 mm	20 sec.

Minium intensity 1200 mW/cm ² light		
Shade	Incremental Depth	Cure Time
A1, A2, A3, A3.5, A4, B1, B2, C2, TW	2.0 mm	10 sec.
A02, A03	1.5 mm	10 sec.

7. Finishing: Remove excess material with suitable finishers or fine diamonds after polymerization. Remove proximal excess with diamond or tungsten carbide finishers, finishing strips, or flexible finishing discs.
8. Adjust occlusion: Check the occlusion apply appropriate grinding corrections to prevent premature contacts or undesired articulation paths on the surface of the restorations.
9. Polishing: Use silicone polishers as well as polishing discs and polishing strips to polish the restorations to durable high gloss.

Caution

1. NANOHYBRID COMPOSITE+ is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.
2. NANOHYBRID COMPOSITE+ contains methacrylic monomer. Some patients can cause allergic reaction to monomer.
3. Do not use NANOHYBRID COMPOSITE+ for patients allergic or hypersensitive to methacrylic monomers.
4. If NANOHYBRID COMPOSITE+ causes an allergic reaction or oversensitivity, discontinue its use immediately.
5. Avoid contact of NANOHYBRID COMPOSITE+ with eyes, mucosal membrane, skin and clothing.
6. If NANOHYBRID COMPOSITE+ comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
7. Use a rubber dam to protect the throat.
8. DO NOT use NANOHYBRID COMPOSITE+ after the indicated expiration date on the syringe.
9. If refrigerated storage is used, allow product to reach room temperature prior to use.

Storage & Use

1. Do not expose restorative materials to elevated temperatures, or to intense light.
2. Store NANOHYBRID COMPOSITE+ at a temperature between 2 - 27°C
3. Ambient temperatures routinely higher than 27°C/80°F may reduce shelf life.
4. Shelf life at room temperature is 36 months.
5. See outer package for expiration date.
6. Do not store materials in proximity to products containing Eugenol.

Warranty

The manufacturer warrants this product will be free from defects in material and manufacture. The manufacturer MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and the manufacturer's sole obligation shall be repair or replacement of the manufacturer's product.

Limitation of Liability

Except where prohibited by law, the manufacturer will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.
Caution: U.S. Federal Law restricts this device to sale or use on the order of a dental professional.

Disposal

It should be eliminated according to local laws and regulations.

Any serious incident in connection with the device should be notified to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

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